


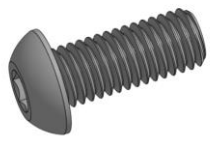

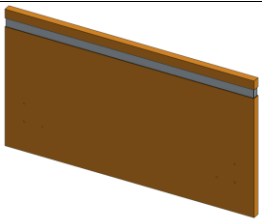
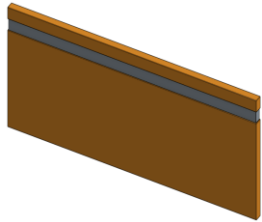

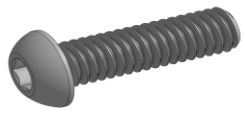
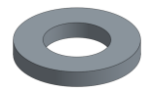
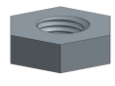


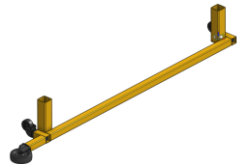






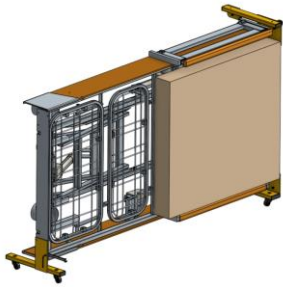
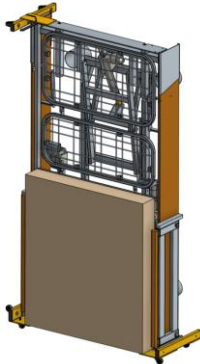
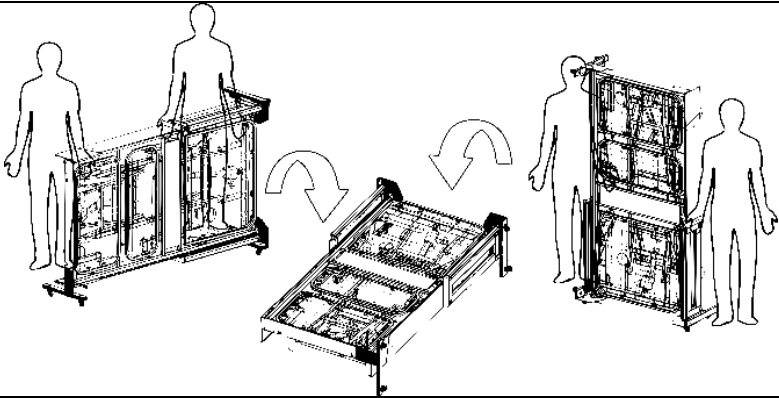
# Installation Instructions

Tools:			4 mm Hex Wrench
			5 mm Hex Wrench
			13mm Wrench
Bed Hardware:	1		M8 x 20mm Button Head Screw (6)
	2		M6 x Joint Connector Screw (6)
Bed Components:			Headboard
			Footboard
			Pendant Holder
Cart Hardware:	A		M8 x 45mm Button Head Screw (4)
	B		M8 Flat Washer (2)
	D		M8 Lock Nut (2)
Cart Components:			Foot End Cart
			Head End Cart (Male)
			Head End Cart (Female)







# Installation Instructions

WARNINGS:		Always Follow Instructions Do Not Discard any Hardware Always disconnect AC power before servicing Bed
		BED IS HEAVY! 2 People Required Improper Handling may result in serious injury
		Always disconnect AC power before servicing Bed
		Refer to owner's manual where appropriate Contact SonderCare if further assistance is required


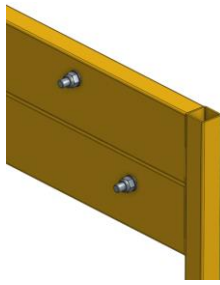
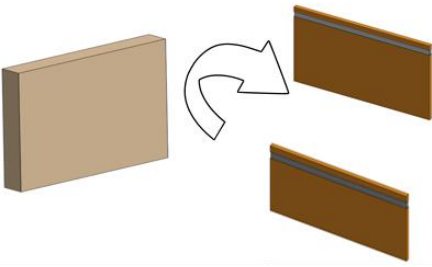
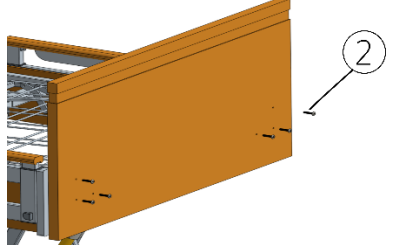
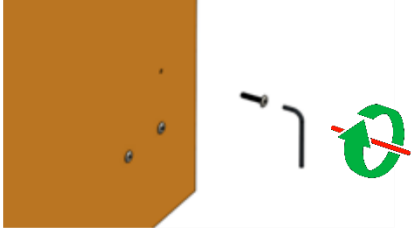
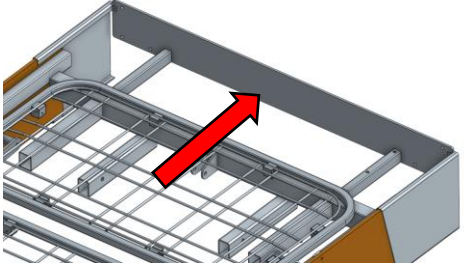

# Installation Instructions

STEP	ACTIVITY		TIMES
1			X 1
<p>Bed may be delivered in the horizontal OR vertical positions            Cut and discard all stretch wrap            Place Head/Footboard box aside for later use            Position bed in a clear open space – if bed is in the vertical position, ensure no overhead obstructions exist</p>			
2			X 1
<p>Using two persons, lower bed from either the horizontal or vertical transport positions to the normal use position            Remove all additional packaging from bed (ALL stretch wrap and bubble wrap)</p>			
<p><b>NOTE:</b></p>	<p>During lowering procedure, support bed from both sides to ensure bed DOES NOT move unintentionally. Failure to do so may result in SERIOUS INJURY.</p>		

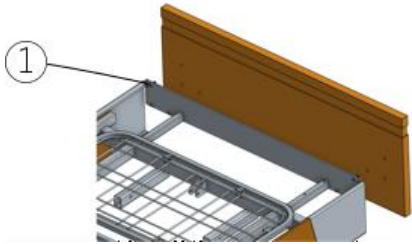
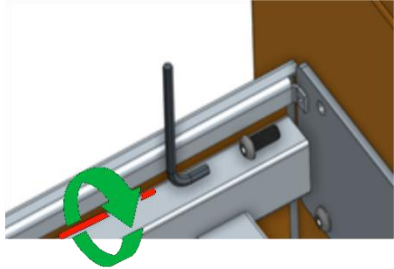
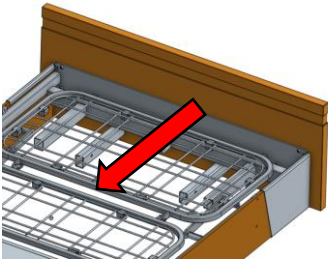

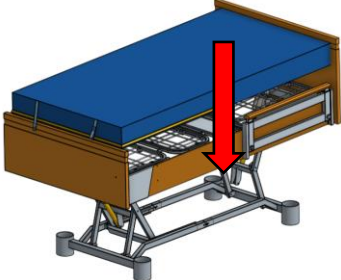
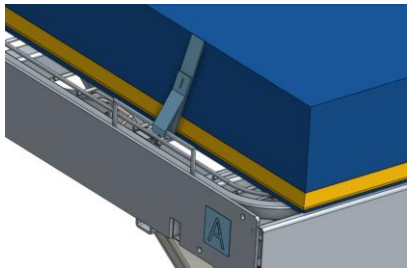
# Installation Instructions

3			X2
<p>While bed is in the normal use position, Remove both M8 x 35mm Button Head Screws from the head end cart Remove Head End Cart (Female) and set aside</p>			
4			X1
<p>Connect bed to an AC power source While standing on the lower cross member of the Head End Cart (Male), raise bed to allow for removal to the Cart</p>			
5			X1
<p>Remove Head End Cart (Male) Attach Head End Cart (Female) to Head End Cart (Male), using removed hardware and store for later use</p>			

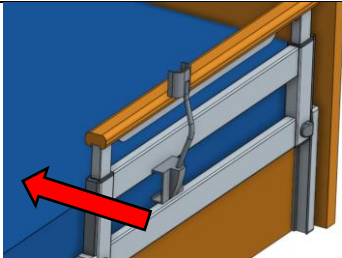
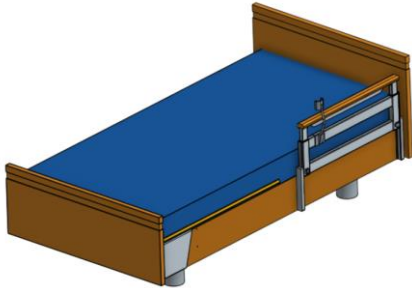

# Installation Instructions

6			X 2
<p>Remove Foot End Cart by using a 13mm wrench and 5mm Hex Key Install hardware back into Foot End Cart and store for later use</p>			
7			X 1
<p>Remove Head/Footboards and hardware package from protective packaging Discard packaging</p>			
8			X 6
<p>Obtain six M6 x Joint Connector Screws Align Headboard with mounting holes on bed frame Insert all screws through Headboard and finger tighten to secure headboard to bed frame Using a 4mm Hex Key, tighten hardware to ensure all fasteners are equally tight and fully seated</p>			
9			X 1
<p>Release foot deck extension locking mechanism (pull up and rotate half turn – both knobs)  Extend foot deck extension to allow for greater working room</p>			

# Installation Instructions

10			X 2
<p>Obtain six M8 x Button Head Screws          Align Footboard with mounting holes on bed frame          Insert all screws through Footboard and finger tighten to secure footboard to bed frame          Using a 5mm Hex Key fashion to ensure all fasteners are equally tight and fully seated</p>			
11			X 1
<p>Collapse foot deck extension          Rotate two knobs and align mating holes           Ensure foot deck extension is locked into place</p>			
12			X 2
<p>Obtain Mattress          Orient mattress with tie down straps at the foot end of the bed          Loop the velcro strap around foot end mattress keeper and back onto itself (Footboard removed for clarity)</p>			

# Installation Instructions

13		X 1
<p>Obtain pendant holder Install pendant holder onto middle section of rail (as shown) Place pendant in pendant holder</p>		
14		X 1
<p>Position bed in the desired location Depress brake pedal at foot end of bed to lock casters  Position bed at desired height Congratulations, bed is now ready to use!</p>		